


Recommended Cameras for the "DELTA 20" using HEINE SLR Photo Adaptor

| Manufacturer | Type | HEINE SLR Photo Adaptor |
|--------------|-------------------|-------------------------|
| Canon | EOS 7D / 50D | K-000.34.185 |
| | EOS 350 D*/400 D* | |
| | EOS 450 D / 500 D | |
| | EOS 1000 D | |
| Nikon | D 40* / D 80* | K-000.34.186 |
| | D 40X | |
| | D 60 / D 90 | |
| | D 200 / D 300 S | |
| | D 700 | |
| | D 3000 / D 5000 | |
| Olympus | E-450 / E-510 | K-000.34.187 |
| | E-520 / E-620 | |

* no longer available

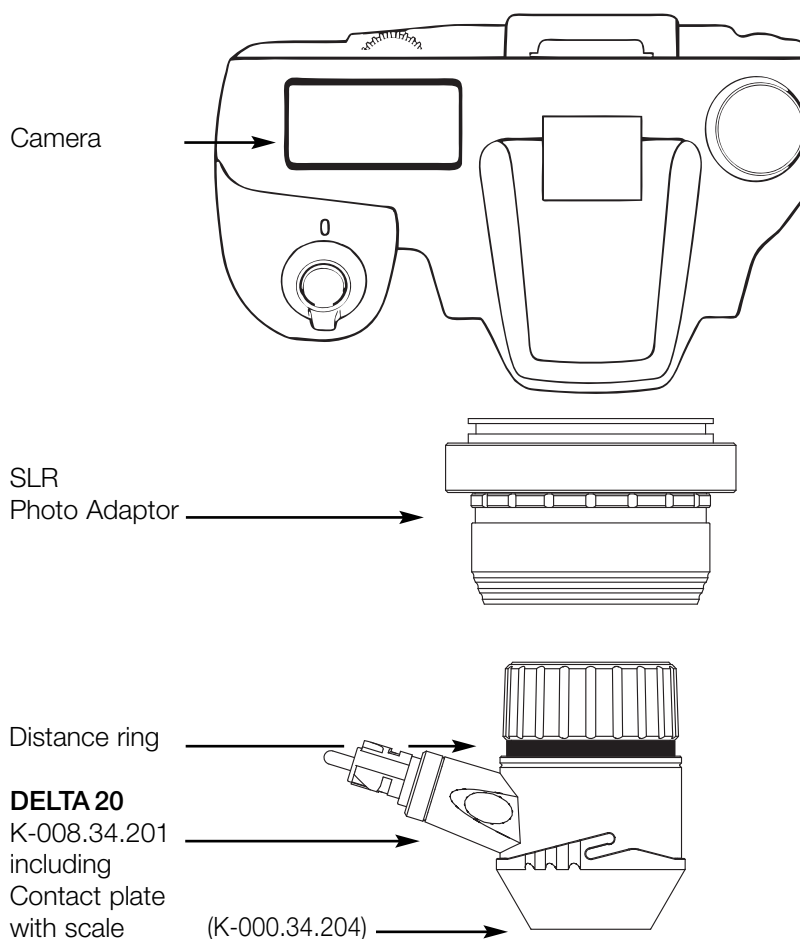

Camera setup for SLR-Adaptor

Nikon

- ISO: 400
- Program: M
- Exposure: 1/40s
- White balance: manual

Canon

- ISO: 400
- Program: Tv
- Exposure: 1/40s
- White balance: manual

Olympus

- ISO: 400
- Program: S
- Exposure: 1/40s
- White balance: manual

Canon Camera Name Chart

| Europe | USA | Japan | MPixel |
|----------------|---------------|--------------|---------------|
| Canon EOS 300D | Digital Rebel | Kiss Digital | 6 |
| 350D | Rebel XT | Kiss N | 8 |
| 1000D | Rebel XS | Kiss F | 10 |
| 400D | Rebel XTi | Kiss X | 10 |
| 450D | Rebel XSi | Kiss X2 | 12 |
| 500D | Rebel T1i | Kiss X3 | 15 |
| 550D | Rebel T2i | Kiss X4 | 18 |